

COMPLIANCE ENABLING ENVIRONMENTAL DISINFECTION

Incidin® OxyFoam & OxyWipe


Ready-to-use cleaner and disinfectant with Hi-speed H_2O_2 ™

The innovative formulation of Incidin OxyFoam & Incidin OxyWipe provides the benefits of a patented* hydrogen peroxide solution with broad spectrum efficacy.

They are effective against a wide range of bacteria and viruses and leave no active residues due to their rapid evaporation rate.

Incidin OxyFoam & OxyWipe provide:

- ▲ Hi-speed H_2O_2 ™
- ▲ EN 16615 compliant
- ▲ Active ingredient decomposes into water and oxygen
- ▲ Excellent material compatibility

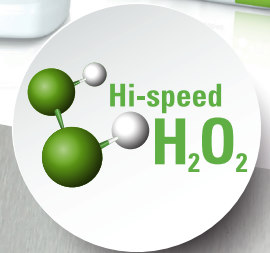

Incidin® OxyFoam & OxyWipe

Ready-to-use cleaner and disinfectant with Hi-speed H₂O₂TM



Bactericidal
(EN 13727 / 16615)



Virucidal against
enveloped viruses



**Noro
virucidal**

Features	Benefits
EFFECTIVE	Broad spectrum efficacy Short contact times
SAFE	Active ingredient decomposes into water and oxygen. Excellent material compatibility
CONVENIENT	Cleans & disinfects in one step Excellent material compatibility

MICROBIOLOGICAL AND VIROLOGICAL TEST RESULTS

EFFICACY UNDILUTED	OxyFoam CONTACT TIME	OxyWipe CONTACT TIME
Disinfection of surfaces acc. to VAH method 2015 (incl. EN 16615), dirty conditions		
Bactericidal, yeasticidal	5 min	15 min
Fungicidal	15 min	15 min
Mycobactericidal	15 min	60 min
Tested acc. RKI recommendation 01/2004*		
Effective against enveloped viruses acc. to RKI recommendation 01/2004*	15 sec	30 sec
Adeno*	1 min	15 min
Polyoma SV40*	30 sec	30 sec
Rota*	30 sec	30 sec
Euro-Norms (clean & dirty conditions)		
EN 13727 (bactericidal)	15 sec	1 min
EN 13624 (yeasticidal)	1 min	15 min
EN 13624 (fungicidal)	5 min	15 min
EN 14348 (tuberculocidal, mycobactericidal)	5 min	30 min
EN 14476 Adeno	30 sec	1 min
EN 14476 Noro	15 min	60 min (dirty conditions)
Euro-Norms (dirty conditions) - 4-field-test		
EN 16615 (bactericidal)	15 sec	5 min
EN 16615 (yeasticidal)	15 sec	1 min
EN 16615 (fungicidal)	5 min	5 min
EN 16615 (tuberculocidal, mycobactericidal)	15 min	60 min

* DVV 2008

INCIDIN OXYFOAM INSTRUCTIONS FOR USE



- 1 Spray surface from ca. 30 cm
- 2 Wipe the surface ensuring it is completely wet
- 3 Let dry



- 1 Apply to clean wipe
- 2 Wipe the surface ensuring it is completely wet
- 3 Let dry

EUROPEAN HEADQUARTERS: ECOLAB EUROPE GMBH

Richtstrasse 7
8304 Wallisellen
Switzerland
+41 (0) 44 877 2001
www.ecolab.com

ECOLAB
DEUTSCHLAND GmbH

Ecolab-Allee 1
40789 Monheim am Rhein
Germany
+49 (0) 2173 599 0
www.de.ecolab.eu

CE 0297

EFFECTIVE.
SAFE.
CONVENIENT.

APPLICATION AREA

Incidin OxyFoam: Cleaning and disinfection foam spray for medical surfaces (incl. e.g. probes) and inventory.



Incidin OxyWipe: Cleaning and disinfection wipes for medical surfaces (incl. e.g. probes) and inventory.



© BIOZID N-12164

INSTRUCTIONS FOR USE

Incidin OxyFoam: Spray surface from ca. 30 cm or apply to clean wipe. Wipe the surface, ensuring the surface is completely wet for the contact time. Or use product undiluted on a clean wipe, wipe the surface and let dry.

Incidin OxyWipe: Take single wipe. Wet surface thoroughly, ensuring contact with the entire surface. Let it dry and do not rinse. Ensure contact with complete surface. Close lid after use.

Not for terminal disinfection of invasive medical devices. Do not use on surfaces sensitive to oxidative agents such as marble, copper or brass.

INGREDIENTS

100g contains the following active:

Incidin OxyFoam: 1.5g

Hydrogen peroxide

Incidin OxyWipe: 1.0g

Hydrogen peroxide.

PACKAGING AND ORDERING INFORMATION

PRODUCT	OUTER SIZE	ORDER CODE
Incidin OxyFoam	6 x 750ml	3082950
Incidin OxyWipe	6 x 100 wipes	3082360

STORAGE

Product should be stored in the original container below 25°C in a cool, dry location out of direct sunlight.

Shelf life after first opening:

Incidin OxyFoam: End of expiry date

Incidin OxyWipe: 1 month

References

1. Incidin OxyFoam (EN 13727 / EN 16615 / EN 14476)

USE BIOCIDES SAFELY. ALWAYS READ THE LABEL AND PRODUCT INFORMATION BEFORE USE. FOR PROFESSIONAL USE ONLY.

ECOLAB